### WHO Emergency Use Assessment and Listing Procedure for EVD IVDs PUBLIC REPORT

### Product: Liferiver<sup>™</sup> - Ebola Virus (EBOV) Real Time RT-PCR Kit Number: EAE 0432-139-00

#### Abstract

In order to respond to the urgent need for quality-assured in vitro diagnostic tests in the event of Ebola Virus Disease (EVD) outbreak, WHO has established a WHO Emergency Use Assessment and Listing (EUAL) Procedure of In Vitro Diagnostics (IVDs) for EVD. It consists of review of existing evidence of safety and performance; desktop review of selected manufacturing and quality management systems documentation and limited laboratory evaluation of the product.

Liferiver<sup>™</sup> - Ebola Virus (EBOV) Real Time RT-PCR Kit with product code QR-0220-02 manufactured by Shanghai ZJ BioTech Co., Ltd., N°20 Building, 528 Ruiquing Road, Zhangjiang High-tech Industrial East District, 201203 Shanghai, People's Republic of China was listed as eligible for WHO procurement 27 April 2015. This pubic report was amended on 22 January 2019 to reflect the inclusion of the latest Instructions for Use.

Liferiver<sup>™</sup> - Ebola Virus (EBOV) Real Time RT-PCR Kit Kit is an in vitro diagnostic test for the detection of all high pathogenic members of Ebolavirus: Zaire ebolavirus (ZEBOV), Sudan ebolavirus (SUDV), Taï Forest ebolavirus (TAFV) and Bundibugyo ebolavirus (BDBV) in blood, serum or plasma (non-heparin anticoagulant). It is intended to aid in the diagnosis of Ebola virus disease (EVD), formerly known as Ebola haemorrhagic fever, among symptomatic individuals. The results should be interpreted strictly according to the Interpretation of Results in this manual. Final judgment should be made with combination of other diagnostic methods. This test is based on real time RT-PCR technology utilizing reverse-transcriptase (RT) reaction to convert RNA into complementary DNA (cDNA). This kit is manually operated and to be used with the RNA extraction kits and PCR instruments listed below by trained laboratory personnel who are proficient in performing real-time RT-PCR assay.

The test is based on real-time RT-PCR technology, utilizing reverse transcriptase (RT) reaction to convert RNA into complementary DNA (cDNA), polymerase chain reaction (PCR) for the amplification of specific target sequences and target specific probes for the detection of the amplified DNA. The probes are labelled with fluorescent reporter and quencher dyes. Probes specific for Ebola virus RNA are labelled with the fluorophore FAM. The probe specific for the target of the Internal Control (IC) is labelled with the fluorophore VIC.

The test consists of three processes:

- Reverse transcription of target RNA to cDNA
- PCR amplification of target cDNA and Internal Control
- Simultaneous detection of PCR amplicons by fluorescent dye labelled probes

Liferiver<sup>™</sup> - Ebola Virus (EBOV) Real Time RT-PCR Kit is validated to be used with

- QIAamp DSP Virus Spin Kit (Qiagen, Cat. No. 61704)
- QIAamp Viral RNA Mini Kit (Qiagen, Cat. No. 52904 or 52906)
- RNA Isolation Kit (Paramagnetic Beads Column) (Liferiver, Cat. No. ME-0010)
- Ribonucleic Acid (RNA) Isolation Kit (Preloaded for Auto-Extraction) (Liferiver, Cat.No. ME-0014)

Liferiver<sup>™</sup> - Ebola Virus (EBOV) Real Time RT-PCR Kit was developed and validated to be used with the following real-time PCR instruments:

- ABI Prism<sup>®</sup> 7500 SDS and 7500 Fast SDS (Applied Biosystems)
- LightCycler<sup>®</sup> 480 Instrument II (Roche)
- CFX96 system/Dx real-time system (Bio-Rad)
- SLAN<sup>®</sup>-96

Liferiver<sup>™</sup> - EBOV Real Time RT-PCR Kit consists of:

- EBOV Super Mix
- RT-PCR Enzyme Mix
- Internal Control (IC)
- EBOV Negative Control
- EBOV Positive Control

#### Test Kit Components

Component	EBOV Super Mix	RT-PCR Enzyme mix	Internal Control	Positive Control Target	Negative Control
Number of Vials	1	1	1	1	1
Volume [μl/Vial]	513	27	30	200	200

EBOV Super Mix and RT-PCR Enzyme mix contain all components (buffer, enzymes, primers, and probes) to allow reverse transcription, PCR mediated amplification and target detection (Filovirus specific RNA and Internal Control) in one reaction setup.

**Storage**: All reagents should be stored at -20±5°C.

#### Background information

Shanghai ZJ Bio-Tech Co., Ltd. submitted an expression of interest for WHO emergency quality assessment of Liferiver<sup>™</sup> - Ebola Virus (EBOV) Real Time RT-PCR Kit on 16 October 2014.

#### 1. Product dossier assessment

Shanghai ZJ Bio-Tech Co., Ltd. submitted documentation in support of safety and performance for Liferiver<sup>™</sup> Ebola Virus (EBOV) Real Time RT-PCR Kit as per the "Invitation to Manufacturers of Ebola Virus In Vitro Diagnostics to Submit an Expression of Interest (EOI) for Emergency Assessment by WHO".<sup>1</sup> The information submitted in the product application was reviewed by WHO staff and external experts (reviewers) appointed by WHO. The findings of the reviews were reported in accordance with "WHO Emergency Use Assessment and Listing Procedure of In Vitro Diagnostics for Ebola Virus Protocol for the Review of Documentary Evidence of Safety, Quality and Performance" (document number WHO PQDx\_0188 v0.2).

Safety and performance documentation assessment conclusion: acceptable.

#### 2. Review of quality management documentation

To establish the eligibility for WHO procurement, Shanghai ZJ Bio-Tech Co., Ltd. was asked to provide up-to-date information about the status of their quality management system.

Based on the review of the submitted quality management system documentation, it was established that sufficient information was provided by Shanghai ZJ Bio-Tech Co., Ltd.to fulfil the requirements described in the "Invitation to manufacturers of Ebola Virus in Vitro Diagnostics to submit an Expression of Interest (EOI) for Emergency Assessment by WHO".

Quality management documentation assessment conclusion: acceptable.

#### 3. Laboratory evaluation

A limited analytical evaluation of the the Liferiver<sup>™</sup> Ebola Virus (EBOV) Real Time RT-PCR Kit was conducted by the Bernhard Nocht Institute for Tropical Medicine (BNITM) in Hamburg, Germany which is a WHO Collaborating Centre for Arbovirus and Haemorrhagic Fever Reference and Research. The limit of detection (LOD) of the assay was verified and compared to the RealStar<sup>®</sup> Filovirus Screen RT-PCR Kit 1.0 (altona Diagnostics GmbH) using simulated specimens generated by spiking cell culture

<sup>&</sup>lt;sup>1</sup> Invitation to manufacturers of Ebola virus in vitro diagnostics to submit an Expression of Interest (EOI) for emergency assessment by WHO. Accessed on 24 November 2014 at

http://www.who.int/diagnostics\_laboratory/141002\_revised\_invitation\_to\_mx\_of\_ebola\_virus\_diagnostics\_rc .pdf?ua=1

supernatants containing infectious Ebola virus strain Makona into whole blood of a healthy donor. The evaluation of the Liferiver™ Ebola Virus (EBOV) Real Time RT-PCR Kit was performed on the Roche LightCycler 480 instrument.

The 95% limit of detection of the assay was found to be 23.9 copies/reaction 95% CI (13.4-405.9 RNA copies/reaction).

Laboratory evaluation conclusion: acceptable.

Based on the review of the submitted documentation and the results of the laboratory evaluation, a recommendation was made to consider Liferiver<sup>™</sup> Ebola Virus (EBOV) Real Time RT-PCR Kit as eligible for WHO procurement.

#### Scope and duration of procurement eligibility

Liferiver<sup>™</sup> Ebola Virus (EBOV) Real Time RT-PCR Kit with product code QR-0220-02 manufactured by Shanghai ZJ Bio-Tech Co., Ltd.is considered to be eligible for WHO procurement. The assay may be used to test symptomatic individuals for EVD. This listing does not infer that the product meets WHO prequalification requirements and does not mean that the product is listed as WHO prequalified.

As part of the on-going requirements for listing as eligible for WHO procurement, Shanghai ZJ Bio-Tech Co., Ltd. must engage in post-market surveillance activities to ensure that the product continues to meet safety, quality and performance requirements. Shanghai ZJ Bio- Tech Co., Ltd is required to notify WHO of any complaints, including adverse events related to the use of the product within 7 days. Furthermore, WHO will continue to monitor the performance of the assay in the field.

WHO reserves the right to rescind eligibility for WHO procurement, if additional information on the safety, quality and performance comes to WHO's attention during post-market surveillance activities.

#### **Commitment to WHO**

As a requirement to listing, the manufacturer is required to participate in the WHO collaborative study for the assessment of the suitability of an interim standard for Ebola virus nucleic acid amplification tests.

Labelling 1. Instructions for Use



Revision No.: ZJ0005 Issue Date: November 16, 2018

CE

IVD

-25°C-

 $\Sigma_{25}$ 

# Ebola Virus (EBOV) Real Time RT-PCR Kit

User Manual

For In Vitro Diagnostic Use Only

REF QR-0220-02

For use with Bio-Rad CFX 96; SLAN<sup>®</sup>-96; ABI Prism<sup>®</sup>7500; LightCycler<sup>®</sup>480 Instruments



Boulevard Général Wahis 53 1030 Brussels, BELGIUM Tel: +(32) 2.732.59.54 Fax: +(32) 2.732.60.03 E-Mail : mail@obelis.net

Obelis S.A.

Shanghai ZJ Bio-Tech Co., Ltd. www.liferiver.com.cn Tel: +86-21-34680596 trade@liferiver.com.cn Fax: +86-21-34680595 2<sup>nd</sup> floor,No.15 Building,No.188 Xinjunhuan Road, PuJiang Hi-tech Park, Shanghai, China

## **Table of Contents**

Com	mon Information	1
1	Intended Use	1
2	Principle of Real Time RT-PCR	1
3	Product Description	2
4	Warnings and Precaution	3
5	Kit Contents	4
6	Storage	4
7	Specimen Requirements	4
7.1	Whole blood	4
7.2	Plasma	4
7.3	Serum	5
8	RNA Extraction Procedure for Specimens and Controls	5
8.1	Experiment Preparation	6
8.1.1	1 Instruments and Materials Preparation	6
8.1.2	2 Specimens Preparation	7
8.2	RNA Isolation	7
8.2.1	1 QIAamp DSP Virus Spin Kit	7
8.2.2	2 QIAGENQIAamp Virus RNA Mini Kit	10
8.2.3	3 Liferiver <sup>™</sup> RNA Isolation Kit (Paramagnetic Beads Column)	13
8.2.4	Liferiver <sup>™</sup> RNA Isolation Kit (Preloaded for Auto-Extraction)	14
8.2.4	4.1 Kit Components and Storage Conditions	15
8.2.4	4.2 RNA Isolation Procedure using the Liferiver <sup>TM</sup> RNA Isolation Kit (Preloaded	l for
Auto	p-Extraction)	15
9	PCR Setup	15
Instr	ument Specific Instructions	17
10	Operation Procedure on Bio-Rad CFX 96	17
11	Operation Procedure on SLAN <sup>®</sup> -96	20
12	Operation Procedure on ABI Prism <sup>®</sup> 7500	25
13	Operation Procedure on LightCycler <sup>®</sup> 480	28
Inter	rpretation of Results	33
14	Interpretation of Controls and Clinical Specimens	33
14.1	Controls	33
14.2	Specimen	33
15	Limitations	34
Anal	ytical Performance Characteristics	35

## **Common Information**

#### 1 Intended Use

Ebola Virus (EBOV) Real Time RT-PCR Kit is an in vitro diagnostic test for the detection of all high pathogenic members of Ebolavirus: Zaire ebolavirus (ZEBOV), Sudan ebolavirus (SUDV), Taï Forest ebolavirus (TAFV) and Bundibugyo ebolavirus (BDBV) in blood, serum or plasma (non-heparin anticoagulant). It is intended to aid in the diagnosis of Ebola virus disease (EVD), formerly known as Ebola haemorrhagic fever, among symptomatic individuals. The results should be interpreted strictly according to the Interpretation of Results in this manual. Final judgment should be made with combination of other diagnostic methods. This test is based on real time RT-PCR technology utilizing reverse-transcriptase (RT) reaction to convert RNA into complementary DNA (cDNA). This kit is manually operated and to be used with the RNA extraction kits and PCR instruments listed below by trained laboratory personnel who are proficient in performing real-time RT-PCR assay.

#### 2 Principle of Real Time RT-PCR

Real Time reverse transcription polymerase chain reaction (Real Time RT-PCR) is used when the starting material is RNA. In this method, RNA is first transcribed into complementary DNA (cDNA) by reverse transcriptase from total RNA. The cDNA is then used as the template for the Real Time PCR.

PCR is a process for amplifying target DNA sequence with thermophilic DNA polymerase. It involves three steps: melting (denaturing of the DNA duplex at a high temperature to yield single stranded DNA), annealing (primers anneal to the single stranded target sequence) and elongation (DNA polymerase extends the primers by adding dNTPs to the phosphate backbone). These steps complete one PCR cycle, and the cycle repeats until a sufficient DNA concentration is reached.

Real Time PCR is regular PCR with the advantage of detecting the amount of DNA formed after each cycle with a fluorescently-tagged oligonucleotide probe. The probeis complementary to the target sequence being amplified. A fluorophore attached to the 5' end of the probe and a quencher dye attached to the 3' end. The fluorescence emitted from fluorophore will be quenched by the quencher dye and so there is no fluorescence can be detected. During each Real Time PCR cycle the probe hybridizes to its target sequence, downstream from a PCR primer. As DNA polymerase extends the primer, it encounters and hydrolyzes the probe from the 5' end, releasing the fluorophore from the probe. The fluorophore is excited and its emission is no longer quenched and there is an increase in fluorescence which can be detected by Rea-Time PCR instrument and generated quantitative data are collected and analyzed by specialized software.

#### **3** Product Description

Ebola Virus (EBOV) is named after the Ebola River Valley in the Democratic Republic of the Congo (formerly Zaire), which is near the site of the first recognized outbreak in 1976 at a mission hospital run by Flemish nuns. It remained largely obscure until 1989 when several widely publicized outbreaks occurred among monkeys in the United States.

The virus interferes with the endothelial cells lining the interior surface of blood vessels and with coagulation. As the blood vessel walls become damaged and destroyed, the platelets are unable to coagulate, patients succumb to hypovolemic shock. Ebola is transmitted through bodily fluids, while conjunctiva exposure may also lead to transmission.

EBOV Real Time RT-PCR kit contains a specialized ready-to-use one step real time RT-PCR system for the detection of EBOV RNA, which includes reverse transcription (RT) for the transcription of EBOV RNA into cDNA and real time PCR for the amplification and detection cDNA from EBOV RNA.

By the end of 2013, 64 Ebola virus strains have been sequenced:

- 46 Zaire ebolavirus sequences and GenBank accession numbers are HQ613403, KF827427, JQ352763, EU224440, AF086833, AY354458, AY142960, AF499101, AF272001, L11365, J04337, HQ613402, JA489037, HC069241, HC069239, HC069235, HC069221, HC069217, HC874661, NC002549, HC874677, HC874681, HC874683, HC874665, HC874667 and HC069223;
- 14 Sudan ebolavirus sequences and GenBank accession numbers are FJ968794, KC242783, EU338380, JN638998, KC545392, KC545391, KC545390, KC545389, AY729654, KC589025, AF173836, HC918462, HC918459 and HC874655;
- 2 Taï Forest ebolavirus sequences and GenBank accession numbers are FJ217162 and JA489027;
- 6 Bundibugyo ebolavirus sequences and GenBank accession numbers are FJ217161, KC545393, KC545394, NC\_014373, KC545395 and JA489018.

All sequences above have been covered by the primers and probes in this kit.

The latest Ebola virus sequence update was on October 14, 2014 and 108 sequences were added. All of them were Zaire ebolavirus and GenBank accession numbers are KC242789, KC242788, KC242787, KC242786, KC242784, KC242790, KC242785, KF990214, KF990213, KC242801, KC242799, KC242794, KC242796, KC242792, KC242791, KC242793, KM655246, KC242800, KJ660348, KJ660347, KJ660346, KM233116, KM233115, KM233114, KM233113, KM233112, KM233110, KM233109, KM233107, KM233106, KM233105, KM233097, KM233096, KM233095, KM233093, KM233092, KM233099, KM233098, KM233088, KM233085, KM233084, KM233081, KM233080, KM233077, KM233075, KM233074, KM233073, KM233072, KM233070, KM233069, KM233067, KM233066, KM233065, KM233064, KM233063, KM233062, KM233061, KM233057, KM233056, KM233055, KM233054, KM233053, KM233052, KM233051, KM233050, KM233049,

KM233047, KM233046, KM233045, KM233044, KM233043, KM233042, KM233041, KM233040, KM233039, KM233038, KM233037, KM233036, KM233035, KM034562, KM034561, KM034560, KM034559, KM034558, KM034557, KM034556, KM034555, KM034554, KM034553, KM034551, KM034550, KM034549, KM233087, KM233086, KM233079, KM233076, KM233048, KM233071, KM233058, KM233059 and KM233082.

All sequences above have been covered by the primers and probes in this kit.

A positive control, a negative control and an internal control (IC) are included in this kit to identify false negative results, false positive results, low extraction efficiency and possible PCR inhibitors. Pseudovirus vectors of four species were constructed as positive control, each of which included one partial Ebola virus sequence respectively. The insert sequences are 1-500 nt of Zaire ebolavirus (KJ660347), 1-500 nt of Sudan ebolavirus (EU338380), 1-500 nt of Taï Forest ebolavirus (JA489027) and 1-500 nt of Bundibugyo ebolavirus (FJ217161).

### 4 **AWarnings and Precaution**

- Carefully read this instruction before starting the procedure.
- For in vitro diagnostic use only.
- This assay needs to be carried out by skilled personnel.
- Specimens should be regarded as potentially infectious materials and handled in accordance with the guidelines recommended by the World Health Organization (<u>http://www.who.int/csr/resources/publications/ebola/laboratory-guidance/en/</u>) or the Centers for Disease Control and Prevention (<u>https://www.cdc.gov/vhf/ebola/laboratory-personnel/specimens.html</u>).
- This assay needs to be run according to Good Laboratory Practice.
- Do not use the kit after its expiration date.
- Avoid repeated freeze-and-thaw of reagents, which may reduce the sensitivity of the test.
- Once the reagents have been thawed, vortex and centrifuge briefly the tubes before use.
- Prepare Master Mix on ice or in cooling block quickly.
- Use separated and segregated working areas for (i) sample preparation, (ii) reaction setup and (iii) amplification/detection activities. The workflow in the laboratory should proceed in unidirectional manner. Always wear disposable gloves in each area and change them before entering a different area.
- Pipets, vials and other working materials should not circulate among working units.
- Use always sterile pipette tips with filters.
- Wear separate coats and gloves in each area.
- Avoid aerosols.

### 5 Kit Contents

Ref.	Kit Contents	Volume
1	EBOV Super Mix	1 vial, 513µl
2	RT-PCR Enzyme Mix	1 vial, 27µl
3	Internal Control (IC)	1 vial, 30µl
4	<b>EBOV Negative Control</b>	1 vial, 200µl
5	<b>EBOV Positive Control</b>	1 vial, 200µl

### **Control materials**

- EBOV Negative Control is saline that will serve as an external negative specimen during RNA extraction procedure.
- EBOV Positive Control is a pseudovirus containing partial EBOV RNA fragment that will serve as an external positive control during RNA extraction procedure. The EBOV RNA fragment in pseudovirus is designed to cover the target sequence to react with the Real Time RT-PCR reagents in this kit to indicate whether the Real Time RT-PCR worked.
- Internal Control (IC) is a pseudovirus containing non-target RNA fragment that will be added into the specimen before RNA extraction procedure as internal control to evaluate RNA extraction efficiency and identify possible PCR inhibitors. The RNA fragment in peudovirus will be amplified by the primers used to amplify target sequence, but it will be detected by another probe.

### 6 Storage

- All reagents should be stored at -20±5°C. Storage at +4°C is not recommended.
- All reagents should be used before expiration date indicated on kit.
- Repeated thaw-freeze cycles should be avoided more than 3 times, as this may reduce the sensitivity of the assay.

## 7 Specimen Requirements

#### 7.1 Whole blood

Take 2ml venous blood with a sterile syringe. The blood is collected in a disposable sterile tube with anticoagulant (EDTA).

#### 7.2 Plasma

Take 2ml venous blood with a sterile syringe. The blood is collected in a disposable sterile anticoagulant tube (EDTA). Separate plasma from blood by centrifugation at 3000rpm for 10 minutes.

#### 7.3 Serum

Take 2ml venous blood with a sterile syringe. The blood is collected in a disposable sterile tube. After coagulation of blood, centrifuge the blood at 3000rpm for 10 minutes

#### Notes:

Suspected infectious blood, plasma or serum must be manipulated at a BSL-4 laboratory before virus inactivation. Four commercial kits are recommended for EBOV RNA extraction, and related inactivation is listed as follows.

Volume of specimen	Treatment	Kits	Cat.No.
200µl	Mix with 25μl QIAGEN Protease (QP), and then add 200 μl Buffer AL (containing 28 μg/ml of carrier RNA). (See 8.2.1 step1-3)	QIAamp DSP Virus Spin Kit (Qiagen)	61704
140µl	Mix with 560µl Buffer AVL–carrier RNA. (See 8.2.2 step1-2)	QIAamp Viral RNA Mini Kit ( Qiagen)	52904/ 52906
140µl	Mix with 526µl Binding solution (See 8.2.3 step1-3)	RNA Isolation Kit (Paramagnetic Beads Column) ( Liferiver)	ME-0010
200µl	Pipette into a row A well of Preparing Plate (containing 500µl binding buffer, 6µl Carrier RNA and 20µl RNA binding Beads) (See 8.2.4 step1-3)	Ribonucleic Acid (RNA) Isolation Kit (Preloaded for Auto-Extraction) (Liferiver)	ME-0014

It is necessary to use a biosafety cabinet to open blood collection tubes with specimens and extract nucleic acid with the commercial kits mentioned above at a BSL-3 laboratory or BSL-2 laboratory. All the inactivated specimens can be stored at 2-8  $^{\circ}C$  for 72 hours. The samples must be stored at -70  $^{\circ}C$  for long term storage. Additionally, multiple freeze/thaw should be avoided.

Inactivated specimens should still be considered as biohazard materials due to the risk that complete inactivation of Ebola specimens has not been demonstrated.

### 8 RNA Extraction Procedure for Specimens and Controls

The nucleic acid extraction reagents are not supplied in this kit and four commerical kits are recommended for EBOV RNA extraction:

- 1) QIAamp DSP Virus Spin Kit (Qiagen, Cat. No. 61704);
- 2) QIAamp Viral RNA Mini Kit (Qiagen, Cat. No. 52904 or 52906);
- 3) RNA Isolation Kit (Paramagnetic Beads Column) (Liferiver, Cat. No. ME-0010);
- 4) Ribonucleic Acid (RNA) Isolation Kit (Preloaded for Auto-Extraction) (Liferiver, Cat.No. ME-0014).

It is noted that the EBOV positive control and EBOV negative control in this kit should be extracted with the same protocol for specimens. The internal control in this kit should be added into the extraction mixture with  $1 \mu l/test$  to monitor the whole process.

### 8.1 Experiment Preparation

#### 8.1.1 Instruments and Materials Preparation

Instruments and materials required for extraction but not provided are listed as below:

- Refrigerator and Freezer ,2-8°C
- Vortexer
- Microcentrifuge<sup>+</sup> (with rotor for 1.5 ml and 2 ml tubes)
- RNase-free microtubes
- Heating block or water bath for lysis of specimens at 56°C
- Tube racks
- Class II biosafety cabinet (or glove box)
- Personal protective equipment: Powder-free gloves, lab coat, eye protection, etc.
- Personal protective equipment: Powder-free gloves, fit-tested masks such as N95 Respirators and Filtering Face Piece (FFP) 3, Powered Air Purifying Respirators (PAPR) if fit-testing fails, full face shields, and disposable impermeable gowns, etc. For specific requirements, please refer to the WHO interim guideline on laboratory diagnosis of Ebola virus disease (<u>http://www.who.int/csr/resources/publications/ebola/laboratory-guidance /en/</u>).
- Pipets<sup>+</sup> and pipette tips (to prevent cross-contamination, we strongly recommend the use of pipet tips with aerosol barriers)
- Ethanol (96–100%)\*
- QIAamp DSP Virus Spin Kit (Cat. No. 61704) or QIAamp Viral RNA Mini Kit (Qiagen, Cat. No. 52904 or 52906) or RNA Isolation Kit (Paramagnetic Beads Column) (Liferiver, Cat. No. ME-0010) or Ribonucleic Acid (RNA) Isolation Kit (Preloaded for Auto-Extraction) (Liferiver, Cat. No. ME-0014) associated with EX2400 Automated Nucleic Acid Extraction system (Liferiver, Cat. No. IE-0001)

#### Notes:

\* Do not use denatured alcohol, which contains other substances such as methanol or Methylethylketone.

<sup>+</sup> To ensure that specimens are properly processed according to the Kit procedures, we strongly recommend that instruments (e.g., pipettes and heating blocks) are calibrated according to the manufacturers' recommendations.

#### 8.1.2 Specimens Preparation

Appropriate specimens are inactivated whole blood, plasma or serum specimens. Non-inactivated specimens should be handled at the highest biosafety level before extraction<sup>1</sup>. After collection and centrifugation, plasma or serum can be stored at 2–8°C for up to 6 hours. For long-term storage, freezing at -20°C or -80°C in aliquots is recommended. Frozen plasma or serum specimens must not be thawed more than once. Repeated freeze-thaw leads to denaturation and precipitation of proteins, resulting in reduced viral titers and therefore reduced yields of viral nucleic acids. In addition, cryoprecipitates might be formed during freeze-thaw. If cryoprecipitates are visible, they can be pelleted by centrifugation at approximately 6,800 x g for 3 min. The cleared supernatant should be removed and processed immediately without disturbing the pellet.

### 8.2 RNA Isolation


#### 8.2.1 QIAamp DSP Virus Spin Kit


- 1) Pipette 25 µl QIAGEN Protease (QP) into a lysis tube (LT).
- Add 200 µl of specimen, or EBOV positive control or EBOV negative control into the lysis tube (LT), add 1 µl of internal control into each LT.
- Add 200 µl Buffer AL (containing 28 µg/ml of carrier RNA). Close the cap and mix by pulse-vortexing for ≥15 sec.
- 4) Incubate at  $56^{\circ}C \pm 3^{\circ}C$  for 15 min  $\pm 1$  min in a heating block.
- 5) Briefly centrifuge the lysis tube (LT) to remove drops from the inside of the lid.
- 6) Add 250 µl ethanol (96–100%) to the tube, close the lid, and mix thoroughly by pulse-vortexing for ≥15 sec. Incubate the lysate with the ethanol for 5 min ± 30 sec at room temperature (15–25°C).
- 7) Briefly centrifuge the tube to remove drops from the inside of the lid.
- 8) Carefully apply all of the lysate from step 7 onto the QIAamp MinElute column without wetting the rim. Close the cap and centrifuge at approximately 6000 x g for >1 min. Place the QIAamp MinElute column in a clean 2 ml wash tube (WT), and discard the wash tube containing the filtrate.
- 9) Carefully open the QIAamp MinElute column, and add 500 µl Buffer AW1 without wetting the rim. Close the cap and centrifuge at approximately 6000 x g for ≥1 min. Place the QIAamp MinElute column in a clean 2 ml wash tube (WT), and discard the wash tube containing the filtrate.
- 10) Carefully open the QIAamp MinElute column, and add 500  $\mu$ l Buffer AW2 without wetting the rim. Close the cap and centrifuge at approximately 6000 x g for >1 min. Place the QIAamp MinElute column in a clean 2 ml wash tube, and discard the wash tube containing the filtrate.

<sup>&</sup>lt;sup>1</sup> Theoretically those recommended extraction kits would be able to inactivate Ebola virus. However, the manufacturer has not verified if Ebola virus would be completely inactivated with those recommended extraction kits due to the inaccessibility of active Ebola virus specimens.

- 11) Carefully open the QIAamp MinElute column and add 500  $\mu$ l ethanol (96 –100%) without wetting the rim. Close the cap and centrifuge at approximately 6000 x g for >1 min. Discard the wash tube containing the filtrate.
- 12) Place the QIAamp MinElute column in a clean 2 ml wash tube (WT). Centrifuge at full speed (approximately 20,000 x g) for 3 min ± 30 sec to dry the membrane completely.
- 13) Place the QIAamp MinElute column into a new 2 ml wash tube (WT), open the lid, and incubate the assembly at 56°C ± 3°C for 3 min ± 30 sec to dry the membrane completely.
- 14) Place the QIAamp MinElute column in an elution tube (ET), and discard the wash tube with the filtrate. Carefully open the lid of the QIAamp MinElute column, and apply 20–150 μl Buffer AVE to the center of the membrane. Close the lid and incubate at room temperature for 5 min.
- 15) Centrifuge at full speed (approximately  $20,000 \times g$ ) for >1 min. RNA is now present in the eluate and ready to test. Store specimens and controls at 2-8°C until PCR master mixes are prepared.
- 16) The RNA solution should be tested by PCR within 6 hours of completing the extraction process. Redundant specimens should be stored at 2-8°C while testing is in progress. Long-term storage of RNA solutions (> 6 hours) should be at -20°C (preferably -80°C). Minimize (not to exceed 3) repeated freeze-thaw cycles.

*Note: For more information, please refer to the manufacturer's instructions.* 

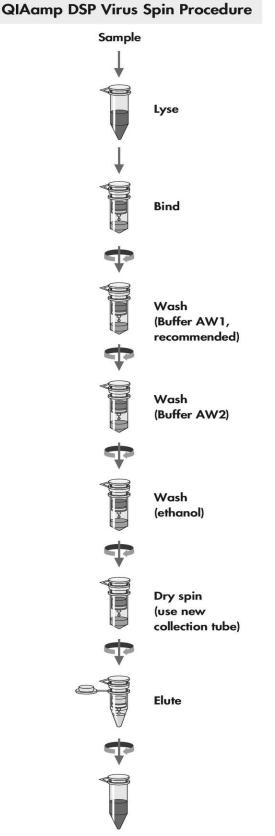

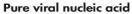

**Figure 1. QIAamp DSF Virus Spin Procedure for Nucleic Acid Isolation.** The illustration shows the simple steps (lyse, bind, wash, and elute) of the extraction protocol.

#### 8.2.2 QIAGEN--QIAamp Virus RNA Mini Kit


Note: Qiagen Buffer AVL with carrier RNA is stable at room temperature for 3-4 hours; however, store reconstituted Buffer AVL/Carrier RNA at 2-8°C for longer periods. A precipitate will form and must be redissolved by warming at 80°C before use. DO NOT warm solution more than 6 times and DO NOT incubate for more than 5 min (see manufacturer's guidelines for more information). It is recommended to aliquot out the Qiagen Buffer AVL containing carrier RNA so as to prevent warming the reagent vial more than 6 times. If aliquoting in larger than single use volumes, then mark on the vial how many times the vial has undergone warming.

- 1) Pipette 560  $\mu$ l of prepared Buffer AVL containing carrier RNA into a 1.5 ml microcentrifuge tube.
- Add 140 μl of specimen, or positive control or negative control and 1 μl internal control to the Buffer AVL-carrier RNA in the microcentrifuge tube. Mix by by pulse-vortexing for 15 sec.
- 3) Incubate at room temperature (15–25°C) for 10 min.
- 4) Briefly centrifuge the tubes to remove drops from the inside of the lid.
- 5) Add 560  $\mu$ l of absolute ethanol (96-100%) to each tube, and mix by pulse-vortexing for 15 sec. After mixing, briefly centrifuge the tubes to remove drops from inside the lid.
- 6) For each specimen and control, place a QIAamp spin column into a 2 ml collection tube (from the QIAamp Viral RNA Mini Kit). Be sure to label the top of the columns clearly.
- 7) Carefully transfer the mixture from step 5, including any precipitate, to the QIAamp spin column WITHOUT wetting the rim of the column.
- 8) Centrifuge 1-2 min at 6,000 x g. If the specimen has not cleared the filter after the first run, repeat centrifugation until the specimen has cleared the filter.
- 9) For each specimen and control, place the QIAamp spin column into a sec, clean
   2 ml collection tube (from the QIAamp Mini Kit) and add 500 μl of Buffer AW1.
   Discard the tube containing the filtrate from the previous step.
- 10) Centrifuge 1-2 min at 6,000 x g. If the buffer has not cleared the filter after 1-2 min, repeat centrifugation until buffer has cleared the filter.
- Place each QIAamp spin column into a third clean 2 ml collection tube (from the QIAamp Mini Kit). Carefully open the QIAamp spin column and add 500 μl of Buffer AW2.
- 12) Centrifuge at full speed (approx.  $14,000 \times g$ ) for 3 min. Discard the tube containing the filtrate from the previous step.
- 13) To eliminate any possible Buffer AW2 carryover, place the QIAamp spin column into a new collection tube, discard the old collection tube, and centrifuge at full speed (approx. 14,000 x g) for 1 min.
- 14) Place the QIAamp Mini column in a clean, clearly labeled 1.5 ml RNase-free microcentrifuge tube (not provided). Discard the old collection tube containing the filtrate.
- 15) Carefully open the QIAamp Mini column and add 60  $\mu$ l of Buffer AVE that has

been equilibrated to room temperature. Close the cap, and incubate at room temperature for 1 min.

- 16) Centrifuge at 6,000 x g for 1 min. RNA is now present in the eluate and ready to test. Store specimens and controls at 2-8°C until PCR master mixes are prepared.
- 17) Extracted specimens should be tested by PCR within 6 hours of completing the extraction process. Redundant specimens should be stored at 2-8°C while testing is in progress. Long-term storage of extracted specimens (> 6 hours) should be at -20°C (preferably -80°C). Minimize (not to exceed 3) repeated freeze-thaw cycles.

*Note: More information please refer to the manufacturer's instruction.* 

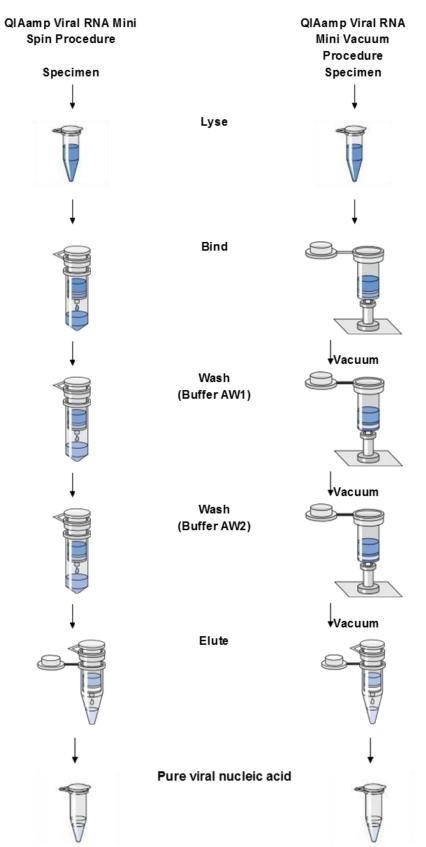

**Figure 2. Viral RNA Isolation Procedure using QIAamp Viral RNA Mini Kit.** The illustration shows the simple steps (lyse, bind, wash, and elute) of the spin protocol and the vacuum protocol.

## 8.2.3 Liferiver<sup>™</sup> RNA Isolation Kit (Paramagnetic Beads

### Column)

- Pipette 526 μl of prepared Binding Solution containing 20 μl of magnetic beads suspension and 6 μl of carrier RNA into each 1.5ml RNase-free tube.
- Transfer 140 µl of specimen, or EBOV positive control or EBOV negative control and 1 µl of internal control to the Binding Solution in the RNase-free tube.
- Mix by vortexing gently for 10 sec or by inverting the tube 5~10 times and incubate at room temperature for 3 min
- 4) Transfer the 667 µl liquid in step 3 to a Binding Column
- 5) Centrifuge the tube at 16000 × g for 60 sec. Discard the filtrate in the collection tube.

Note: In some cases, leakage of beads through the membrane would be observed. It produces no significant effect on the extraction efficiency.

- 6) Add 500  $\mu$ l of Washing Buffer A to each specimen and centrifuge the tube at 16000 × g for 40 sec. Discard the filtrate in the collection tube.
- 7) Repeat step 6.
- 8) Add 500  $\mu$ l of Washing Buffer W to each specimen and centrifuge the tube at 16000 × g for 15 sec. Discard the liquid in the collection tube.
- 9) Repeat steps 8.
- 10) Centrifuge the binding columns at  $16000 \times g$  for 2min to thoroughly dry the membrane.
- Add 50 μl of Elution Buffer (preheated to 65°C) to each column, and incubate for 2 min at room temperature.
- 12) Centrifuge the tube at 16000 × g for 2min, and store the purified RNA in the 1.5 ml RNase-free tube at -20 °C.

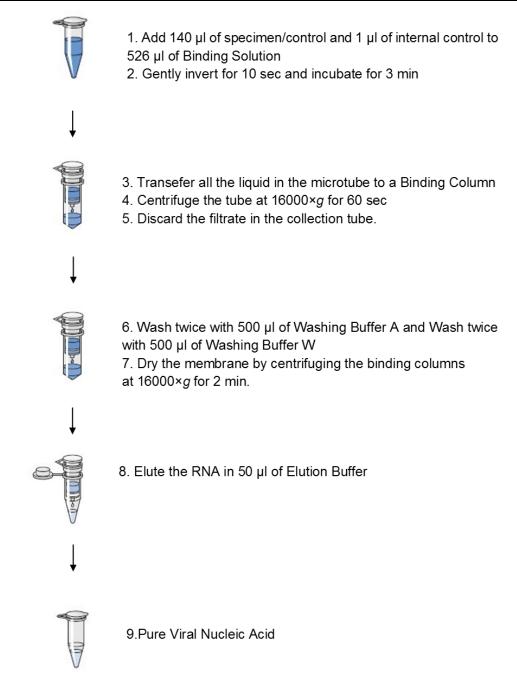


Figure 3. Viral RNA Isolation Procedure using Liferiver<sup>™</sup> RNA Isolation Kit (Paramagnetic Beads Column). The illustration shows the simple steps (lyse, bind, wash, and elute) of the protocol.

## 8.2.4 Liferiver<sup>™</sup> RNA Isolation Kit (Preloaded for

#### Auto-Extraction)

EBOV nucleic acid extraction can be fully automated using the Liferiver RNA isolation Kit (Preloaded for Auto-Extraction) on the EX2400 Automated Nucleic Acid Extraction system (Liferiver, Cat. No. IE-0001).

Ref.	Component	Amount	Storage	Volume/test
1	Preparing Plate <b>a</b>	20 pieces	room temp	
2	Magnetic Cap	20 strips	room temp	
3	Elution Buffer	1ml×2	room temp	
4	Carrier RNA <b>b</b>	5 tubes	room temp	6µl
		(add 350µl Elution Buffer to		
		each tube before use)		

### 8.2.4.1 Kit Components and Storage Conditions

Note:

**a** Do not make Preparing Plate frozen. Wells in Row A of preparing plate tend to be crystallized at low temperature, so dissolve it at 37 °C before use.

**b** Once Elution Buffer has been added to Carrier RNA, store the Carrier RNA at 2 °C-8 °C for 6 months or at-20 °C for 24 months.

Properly stored kits are guaranteed for 24 months from the date manufactured.

### 8.2.4.2 RNA Isolation Procedure using the Liferiver<sup>™</sup> RNA

### Isolation Kit (Preloaded for Auto-Extraction)

- 1) Pipette 1  $\mu I$  EBOV internal control into each well in row A.
- 2) Take out one piece of preparing plate and tear the aluminum foil carefully.
- 3) Add 6 $\mu$ l Carrier RNA and 200  $\mu$ l mix (specimen, EBOV positive control or EBOV negative control and EBOV internal control) into the well A1-A12 of the preparing plate.
- 4) Put the preparing plate on the transport platform carefully, insert the magnetic cap.
- 5) Choose "RNA Isolation" program, press "START" to run the test.
- 6) After the program finished, discard the magnetic cap, take out the preparing plate and transfer the liquid in well E1-E12 into RNase-free EP tubes. It can be used for immediate experiment or store at -20 °C for preservation.

## 9 PCR Setup

8.1 Determine the number of reactions (N) that equals to sample number (n) including positive and negative control plus 1 (N=n+1) due to pipetting error.Prepare EBOV Master Mix by mixing EBOV Super Mix and RT-PCR Enzyme Mix in a 1.5ml centrifuge tube. Required volumes of both reagentsare calculated according the formulation in the table below.

Reagent	Volume of Reagent (N× per reaction)
EBOV Super Mix	Ν×19μl
RT-PCR Enzyme Mix	N×1µl
Total Volume of EBOV Master Mix	N×20µl

\*Number of the reactions (N) includes sample number, control number and one additional reaction due to pipetting error.

- 8.2 Mix the reagent mixture completely by gently pipetting up and down and then spin briefly.
- 8.3 Dispense 20µl Master Mix into each PCR tube. Pipette 5µl sample including positive and negative control into each tube. Close the tubes immediately to avoid possible contamination and spin briefly.

## **Instrument Specific Instructions**

## **10** Operation Procedure on Bio-Rad CFX 96



10.1 Double-click the icon

to open the Bio-Rad CFX software.

10.2 Create a new Experiment and choose the type of the instrument, click "OK".



10.3 Under "Protocol" interface, click the "Create New" button to create a new protocol or click the "Select Existing" button to invoke the existed protocol file.



10.4 After click the "Create New" button, you can see the below interface. Insert a step first.

Protocol Editor - New				×
File Settings Tools				
🔚 🚔 Insert Step After 💌 S	ample Volume 25 µl Est. B	tun Time 01:09:00  ?		
1	2		3	4
2500 C 3:00	95.0 0 0:10	2	55.0 C 0:30	G E 0 N T D 2 39 x
	) C for 0:10 ) C for 0:30 late Read 02 , 39 more times			
				OK Cancel

#### 10.5 Parameters setting:

_	0		
	Step 1	45°C for 10min	1 cycle
	Step 2	95°C for 15min	
	Step 3	95°C for 15sec	40 cycles
	Step 4	60°C for 1min	
		*Fluorescence measured at 60°C	

#### Sample Volume: 25µl

Protocol Editor - N	e <b>v</b>			
<u>F</u> ile Settings Tools	$\frown$			
🔚 🚔 Insert Step 🛛	fter 💙 Sample Volume 25 µl Est.	Run Time 01:51:00 ?		
1	2	3	4	5
45.0 C 10:00	95.0 C 15:00	95.0 C 0:15		G E O N T D O
	1 45.0 C for 10:00			
Insert Step	2 95.0 C for 15:00 → 3 95.0 C for 0:15			
💦 Insert Gradient	4 60.0 C for 1:00			
	+ Plate Read			
Insert GOTO	└── 5 GOTO 3 , 39 more times END			
Insert Melt Curve				
Kemove <u>P</u> late Read				
Step Options				
Nelete Step				
			<u>OK</u>	ncel

10.6 Click the "Ok". Save the protocol.

10.7 Under "Plate" interface, click the "Create New" button to create a new plate or click the "Select Existing" button to invoke the existed plate file.

<u>O</u> ptions							
📶 Protocol	🛄 Plate	•••					
Create <u>N</u> ew							
Select Existing							
-Selected Plat	te						

10.8 Select the fluorescence channel: Click "FAM" and "VIC" channel, click "OK".



Channel	Fluorophore	Selected	Color
1	FAM	2	
	SYBR	Г	
2	HEX	Π	
	TET	Г	
	Cal Gold 540	Г	r.
	VIC	<b>V</b>	
3	ROX		
	Texas Red	Г	
	Cal Red 610	Г	
4	Cy5	Γ	2
	Quasar 670	Г	
5	Quasar 705	Г	

10.9 Choose the well and select sample type in the drop-down list box. Sample choose "Unknown", positive control choose "Positive Control", negative control choose "Negative Control".

late E	Editor - Ne	ew	-	-									
	Setting			1			- 10				1		
	Zoo	m 100%	•	So So	an Mode	All Char	nnels	<u> </u>	🕤 Well G	roups	Trac	e Styles	? 🏩 Plate Loading Guid
4	1	2	3	4	5	6	7	8	9	10	11	12	Select Fluorophores
A													Sample Type
в													Load Unknown Standard NTC
-													FAM     FAM     Fositive Con     Negative Con     Negative Con     NRT     VIC     NRT
n											-		Load Sample Name

10.10 Choose all the wells in the experiment, click "FAM" and "VIC"


File		gs Too									12		
	a zo	om 100%		S IS		e All Cha	nnels	•]  🧯	👌 Well G		🔍 Tra	ce Styles	? 🏨 Plate Loading Guide
A	1 Unk FAM VIC	2 FAM VIC	3	4	5	6	7	8	9	10	11	12	Sample Type
в	Unk FAM VIC	Unk FAM VIC							6				Load Target Name
с	Unk FAM VIC	Pos FAM VIC											VIC VIC (none)
D	Unk FAM VIC	Neg FAM VIC						22 23					Load Sample Name

10.11 Click the "Ok". Save the plate.

10.12 Under "Start Run" interface, click "Open Lid", then add the plate into the instrument, click "Close Lid". Click "Start run". Then save the run file. The experiment begins.

Experiment	Setup				X
Options					
M Protocol	💷 Plate 🅩 Start Ru	n			
Run Informat					
	EBOV.prcl				
Notes :	EBOV.pltd				
Scan Node :	All Channels				
Start Run on	Selected Block(s)				
	Block Name	∆ Type	Run Status	Sample Volume	Protocol ID
CC0174	41	"96FX"	Idle	25	
Select Al		id 🛛 🖍 Close Lid			
Careau p	ilock Open Li	id 🖉 🖉 Close Lid			
					🕨 <u>S</u> tart Bun
					K Prey Hegt >>

10.13 PCR Analysis: after the run is completed, click the well to read the Ct value on the lower right and amplification curve on the upper left.

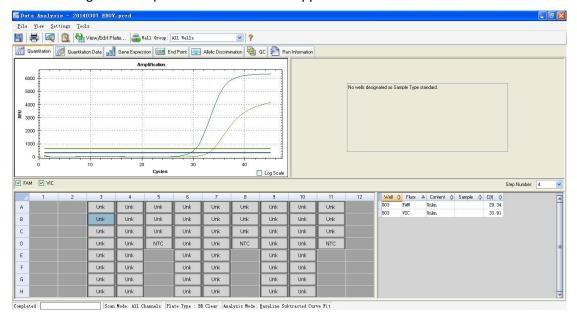

## **11** Operation Procedure on SLAN<sup>®</sup>-96

- 11.1 Enter the SLAN PCR program by double clicking on the SLAN system icon on the desktop.
- 11.2 Select "Creat New Document" from the Quick start up menu. The "New Experiment Wizard" screen will appear.

New Experiment Wiza	rd		<b>X</b>
1. Wizard Type			
Experiment Type	Quantitative/Qualitative/Melting	•	
New Experiment	t		
🔘 New Experimer	t from Template		
		~	
		Ŧ	Browse
		Next	Cancel

11.3 Click "Next", then enter the experiment name and save path in the new interface.

New Experiment Wiza	rd 📃 🔀
1. Wizard Type	2. Information
Experiment Name	Experiment_2014_10_15_14_29_13
Operator	◆ Assessor ◆
Save to	E:\\ Browse
Note	A
	-
Encryption	Previous Next Cancel


11.4 Click "Next". Under temperature control, select "Tube Control", enter the PCR reaction volume:  $25\mu$ l.

New Experiment Wizard	
1. Wizard Type 2. Information 3. Exp	eriment Option 4. Choose Block
Hot-lid Vse Hot-lid	Channel Scan Mode ③ Only Scan Channels In Project ○ Scan All Channels
Temperature Control	
O Block Control	
Tube Control Reaction Vol	Lume(ul): 25 🗳
	Previous Next Cancel

11.5 Click "Next", choose block A and/or block B where the PCR tubes are.

New Experiment Wizard	x							
1. Wizard Type 2. Information 3. Experiment Option 4. Choose Block								
📝 Choose Block A								
📝 Choose Block B								
Previous Confirm Cance	1							

11.6 Click "Confirm". In the new screen, edit project parameters: click "Project"  $\rightarrow$  "Create Project".

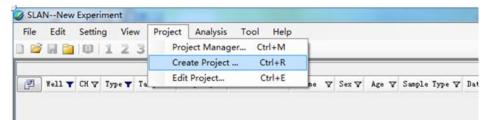

11.7 The below window will appear. Enter the project name and save path, select the channel (FAM & VIC).

Create a new pro	oject					×					
Project Type Quantitative/Qualitative/Melting -											
iioject iype											
Project Name	Project Name Ebola										
Save to C:\Program Files\SLAN全自动医用PCR分析系统 8.3.1\Project\											
	<b>v</b>										
Project Note	2					*					
						*					
-Unit Option											
Unit	Copies/	ml									
Target Informa	tion										
Channel	1	2	3	4	5	6					
Dye	FAM, SYBR	HEX, VIC, JOE	ROX, TEXRD	CY5	N/A	N/A					
Use		✓									
Target Name	FAM	VIC									
J											
Advanced Opti	on			[	Confirm	Cancel					

Segment 1	Step 1	45°C for 10min	1 cycle						
	Step 2	95°C for 15min							
Segment 2	Step 1	95°C for 15sec	40 cycles						
	Step 2	60°C for 1min							
		*Fluorescence measured at 60°C							

11.8 Click "Confirm". Set parameters in the new window.

#### 11.8.1Segment 1: click "Add Step", set the parameters.

<b>: \Program</b> Ho <del>lding and</del> Segment 1			用PCR分析系 /sis Parameter			ject_2014_	<u>10_16_19</u>	55_54.prj		1
tycles 1										Add Segm
	Temp. (°C)	Hold (mm:ss)	Acquisition	Advanced	Temperature Variation	Variation Value(°C)	Time Variation	Variation Value (mm:ss)	Ramp (°C/s)	Delete Segment
Step 1	45.0	10:00			Decrease	00.0	Decrease	00:00	0.0	Segment
Step 2 🥠	95.0	15:00			Decrease	00.0	Decrease	00:00	0.0	
• 11										Add Step Delete St



C:\Program Files\SLAN全自动医用PCR分析系统 8.3.1\Project\Project_2014_10_16_19_55_54.prj 🛛 🛛 🛛													
Holding and cycling Melting Analysis Parameter Rule CrossTalk													
Segment 1 Segment 2													
Cycles 40													
Temp. ("C	) Hold (mm:ss)	Acquisition	Advanced	Temperature Variation	Variation Value (°C)	Time Variation	Variation Value (mm:ss)	Ramp ("C/s)	Delete Segment				
Step 1 95.0	00:15			Decrease	00.0	Decrease	00:00	0.0					
Step 2 🕨 60.0	01:00			Decrease	00.0	Decrease	00:00	0.0					
<]				111				) >	Add Step Delete Step				
Thermal Program													
	Thermal Frogram  Estimated Time: 01:29:53  95.0°C 9												
Segment 1 ×	L I	Segment 2	× 40										
Option Passwor	i							Save	Close				

11.9 Click "Save", then click "Close" to exit.

11.10 Well Edit: select the well where the tubes are, right-click. If are samples, choose "Unknown", if are negative control or positive control, choose "Negative" or "Positive".

	Thermal Program	Temperature Program Temperature
2	g Vali ▼ Of ∇ Type ▼ Target ∇ Property ∇ Labal ∇ Home ∇ Son ∇ Age ∇ Sougle Type ∇ Bate Callected ∇ *	0 0 (S) And (S)
		Running Statu Buning M/A Cycle Times 0 Tesperature 0.0 Mudday Time 00.00 Buning Time 00.00 Buning Time 00.00
	/ Raw Curve / Jorg	Control Mode
		Control Mode
	Well Edit Co/+E	Micaton Gave Control Mode Real California
	Well Edit Ctrl+E	Micaton Gave
	Well Edit CM+E	Stication Carve
	Well Edt         Ctrl+E            1         2         3         4         5         6         7         Undown	Stication Carve
	Weil Edit         Ctrl+E         >           1         2         3         4         5         6         7         Ukleown           1         2         3         4         5         6         7         Ukleown         5           1         2         3         4         5         6         7         Ukleown         5           0         0         0         0         0         7         Ukleown         5           1         0         0         0         0         7         Ukleown         5           0         0         0         0         0         0         7         0           1         0         0         0         0         7         0	Stication Carve
	Weil Edit         Cori+E         -           1         2         3         4         5         6         7         Unknown           Nagative Positive NTC         QC         QC         2         3         3         3         3         3	Stication Carve
	Image: Contract of the state of th	Micaton Gave


11.11 Take a sample well as an example. Click "Unknown", the below screen will appear. Select the project setup in 11.7, click "Confirm".

Well Edit					×
Select	Project: Ebola*	)		•	Browse
Channel	Target Name	Sample Type	Property		
1	FAM	Unknown			
2	VIC	Unknown			
+ Project	Review				
🔲 Set R	eplicates Replicates ID	0	 [	Confirm	Close

11.12 Click block A or block B or block A&B to unlock the hot lid needed. Then open the machine lid to place the sample in.



- 11.13 Start run by clicking "Start Experiment" button.
- 11.14 PCR Analysis: after the run is completed, click the well to read the result. View and adjust the raw data on the right. Selecting "Baseline" and "Thresheld Values", then click 'Analysis'. The Ct value is on the left and the amplification curve on the lower right.

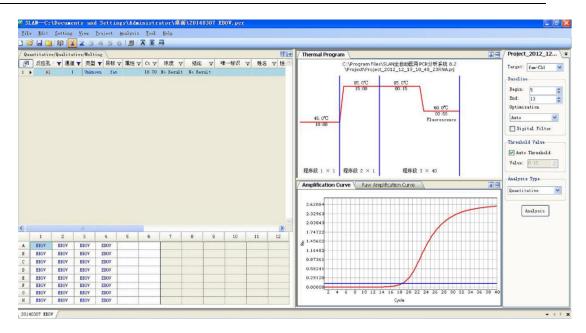


## 12 Operation Procedure on ABI Prism<sup>®</sup>7500

12.1 Double click on the ABI system icon on the desktop to enter the program.

12.2 Select "Create New Document" from the Quick start up menu. The "New Document Wizard" screen will appear. Change the plate name.

New Docum		X
Define Doc Select the comments.	ument assay, container, and template for the document, and enter the operator name and	
Assay:	Standard Curve (Absolute Quantitation)	
Container:	96-Well Clear	
Template:	Blank Document 💌 Browse	
Run Mode:	Standard 7500	
Operator:	Administrator	
Comments:	SIS v1.4	
Plate Name:	Platel	
	,	
	<上─步 (2) 下──步 (2) > 二完成 取消	

12.3 Click "Next", a new screen will appear as below.

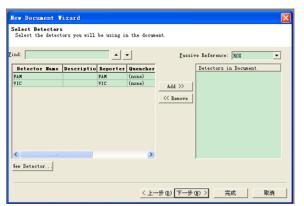

<		
ew Dete	ctor)	

12.4 If there is no "FAM" and "VIC", click the "New Detector"

lew Detector	X
<u>N</u> ame:	
Description:	
<u>R</u> eporter Dye:	F AM
Quencher Dye:	CY3 CY5 FAM
<u>C</u> olor:	JOE NED
N <u>o</u> tes:	ROX SYBR TAMRA
	TEXAS RED VIC
Cr <u>e</u> ate Ar	other OK Cancel

12.5 Select the Detector (FAM & VIC), change "Rox" to "None" in the "Passive Reference" drop-down menu.

name.

Hew Document Vizard	
Select Detectors Select the detectors you will be using in the docume	ent.
Eind: Detector Hame Descriptio Reporter Quencher FAM FAM (none) VIC VIC (none)	Passive Reference: [none]
<u>Sew Detector</u> 	步 (g) 下一步 (g) 〉 完成 取消

12.6 Click "Next", select the well containing the samples and controls, and then click the Detector.

Use		Detecto		B	Reporter Quencher				Task		antity
<u>920</u>	FAX	Detecto	<u> </u>	FAX	eporter	(none		Unkno		44	
P	VIC			VIC		(none		Unkno	WEL		
											>
	1 2	3	4	5	6	7	8	9	10	11	12
V	ע עוע	עוע	עוע	עוע	עוע	ען ע	עע	עןע	עוע	עוע	עוע
V	ນ ນານ	ປັ່ນ	עוע	ປັ່ນ		ប្រ	ប្រ	មប	មម	ប្រ	עוע
U		עע	עע	טע	υυ	עע	νν	υυ	νv	עע	עע
U		0 0	עע	មប	មប	ឋម	បប	υυ	មប	עוע	עוע
V		<u> </u>	UU	υυ	υν	υυ	υυ	VVV	υυ	νv	UU
V	υ υυ	<u> </u>	עע	ປປ	υυ	טע	υυ	עע	ນ ນ	עע	עע
V	ນ ນານ	עוע	עוע	ປັບ	עע	ນ ທີ່ ນ	νν	ນັ້ນ	עע	עוע	עוע
	ע עוע	<u> </u>	שוט	עוע	ושע	ע ע	ע ע	ឋារា	1 11 11	עוע	עוע

12.7 Click "Finish".

12.8 Select the "Instrument" tab. Set up the parameters as follows:


- Stage 1: 45  $^{\circ}$ C for 10 min, 1 cycle;
- Stage 2: 95  $^{\circ}$ C for 15 min, 1 cycle;
- Stage 3: 95  $^\circ\!\! C$  for 15 sec, 60  $^\circ\!\! C$  for 60 sec, 40 cycles.
- Sample Volume: 25 (µl)
- Data Collection at Stage 3, Step 2 (60.0 @ 1:00)

🔯 7500 System SDS Software - [Plate1 (Stand	lard Curve)]	
🙋 File Yiew Icols Instrument Analysis Mindow Help		_ 8 ×
] 🗋 🚔 🖬 🎒 🔃 🔍 🖾 📰 🕨 🖪 🗛 🤶		
/ Setup / Instrument ( Results )		
Instrument Control Start Estimated Time Remaining Stop Disconnect Status:	Temperature Sample: Heat Cover: Block: Cycle Stage: Rep: , Time Step:	
Extend Thermal Cycler Protocol Thermal Profile Auto Increment Ramp Fate Stage 1 Stage 2 St	State:	
Data Collection : Stage 3, Step 2 (60.0 @ 1:00		
Ready	Disconnected	NUM

12.9 Saving the document, after the saving to run the evaluation, click the "Start".

12.10 PCR Analysis: after the run is completed, click the "Results". Click the "Amplification Plot" tab and view and adjust the raw data.

- In the "Data" window, "Delta Rn vs Cycle" should be selected.
- In the "Detector" window, "FAM" and "VIC" should be selected.
- The "Start (cycle)" window should read "6." The "End (cycle)" should be 15.
- Lastly, be sure to click the "Analyze" icon to update the analysis.

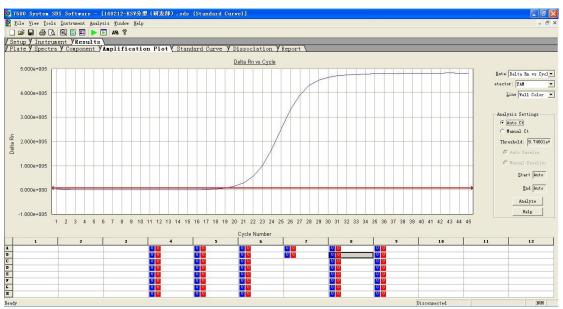

12.11 Click the "Report" icon above the graph to display the cycle threshold (Ct) values.

tup	I 🗇 🖪 🖾 🖾				_		100 C					_
ate												
	Spectra V Com	ponent y	Amplificatio	n Plot y	Standard (	Curve Y Diss	ociation VReport	2				
ell	Sample Hame		Detector		Task	Ct	StdDev Ct	Quantity	Xean Qty	StdDev Qty	Filtered	Te
	684 1:10	FAM		Unknown		19.9803	0. 203					
	684 1:10	AIC		Unknown		Undet.						
						-10						
	1	2	3	4		5	6		9	10	11	12
	1	2	3			5				10	11	12
	1	2	3	UU		5	0 0	<b>1</b>		10	11	12
	1	2	3	UU		5	0 0	<b>1</b>		10	11	12
	3	2	3	U U U V	U 💟	5	0 0			10	11	12
	1	2	3	U U U U U U	U U V V	5 V V				10	11	12
	1	2	3	U U U U U U	U U V V	5 V V				10	11	12
	1	2	3		V V V V V V	5 Ŭ Ŭ Ŭ	V V V V V V			10		12
	1	2	9	VV VV VV VV	U U U U U U U U	5 V V V V	V V V V V V V V V				11	12
	1	2	3	VV VV VV VV	V V V V V V	5 V V V V	V V V V V V V V V			10	11	12
	×	2	3		0 0 0 0 0 0 0 0 0 0 0	5 V V V V V V				10		12
	1	2	3			5 V V V V V V					11	12
	1	2	3		0 0 0 0 0 0 0 0 0 0 0	5 V V V V V V				10	11	12

## **13** Operation Procedure on LightCycler<sup>®</sup>480

13.1 Open the software of LightCycler 480.

13.2 Click "New Experiment" on the right side.

New <u>E</u> xperime		nt	
Experime			
	entiron	n Ma	010
operimer	t from	Temp	olate
			xperiment from Temp pen Existing Object

Show the following interface:

Instrumen	t: Virt	wal LightCy	cler 4	80 96 System I / Not	Connected			Database: My C	omputer (Trac	eable)		Roche
Window:	Ne	w Experime	nt			•		User: Syste	m Admin			Inocito
Experi- ment	Setu		Prot	ocol		Data			Run No	tes		<b>5</b> ]]
$\square$	Dete	ction Forma	t No	no Color Hydro 💌	Customize	Block Siz	<b>e</b> 96	Plate ID		Reaction Vo	lume 20 📫	
Subset Editor	Colo	r Comp ID			Lot No			Tes	t ID			10 <b>7</b>
Sample						Pr	ograms					
Editor	$\bigcirc$	Program     Program		e					Cycle 1	s Ana	Ilysis Mode •	물
Analysis	Θ											
Report	$\mathbf{\nabla}$											
$\equiv$						Temper		· ·				رها
Sum.		Target		Acquisition Mode	Hold (hh:mm:ss	C.	s)	Acquisitions (pe °C)	(°C)	Step Size ( C)	(cycles)	$\land$
$\square$	$\odot$	▶ 95	÷	None	00:00:01	4.4	÷	-		0	0	
	$\Theta \\ \checkmark$											$\overline{\otimes}$
												E.
	ture (	0 (			E:	Overvie timated Ti		14 NSS]				
	Aj	oply oplate	)					End Pro	gram 🔶 + 1	0 Cycles	<u>S</u> tart Run	5

- 13.3 Choose the detection channel:
- 13.3.1Under "Detection Format" option, choose "Dual Color Hydrolysis Probe/UPL Probe".

Run Protocol							
– Setup– Detection Format	Ducl	Calar	Urrely a	Guatamiza			
Color Comp ID	Duar	COIDE	Hydro •	Lot No			
Color Comp in J				LOUND			

13.3.2Click "Customize", and choose "FAM" and "VIC/HEX/Yellow", then click OK



D, F	etection Formats Detection Format Dual Color Hydrolysis Probe / UPL Probe - Integration Time Mode							
	Active	Filter Combination						
Þ	~	FAM (483-533)						
⊢	✓	VIC /HEX / Yellow555 (523-568)						

13.4 Change the "Reaction Volume": 25ul



13.5 Under "Programs" tab, select "Program", 1 Cycles, you can also change the Program Name.

	Programs		
$\square$	Program Name	Cycles	Analysis Mode
A	▶ Program	1	None 🔻
U			

13.6 Under "Program Temperature Targets" tab, add one step , then set the temperature and time parameter: 45 °C, 10 min; 95 °C, 15 min.

		Program Temperature Targets								
$\frown$		Target (°C)	Acquisition Mod	е	Hold (hh:mm:ss)	Ramp Rate (°C/s)	Acquisitions (per °C)	Sec Target (°	Step Size (°C)	Step Delay
$\overline{\mathbf{O}}$				/				C)		(cycles)
Ð	45	-	None		00:10:00	4.4		o 🛟	0 🗘 0	* *
$\overline{\frown}$	<b>)</b> 95	÷	None	-	00:15:00	÷ 4.4 ÷		o 🛟	0 🗘 0	*
$\Theta$			_							

13.7 Go back to "Programs" tab, add a new program , 40 Cycles and choose "Quantification" under the Analysis Mode option.

	Programs		
$\frown$	Program Name	Cycles	Analysis Mode
A	Program	1	None 🔻
$\mathbf{E}$	Program	40 🗘	Quantification 🔹

13.8 Under this new added program, go to the "Program Temperature Targets" tab, and then set the temperature and time parameter:  $95^{\circ}$ C, 15sec;  $60^{\circ}$ C, 60sec, and the "Acquisition Mode" should be chosen Single.

						Pre	ogram Temperatur	e Targets			
	Т	arget (°C)	Acquisit	ion Mode	Hold (hh:	mm:ss)	Ramp Rate (°C/s)	Acquisitions (per °C)	Sec Target (° C)	Step Size (°C)	Step Delay (cycles)
$\odot$	95	~	None	(	00:00:15	Ť	4.4		o 🕂	0 🗘 0	
Θ	▶ 60	- (	Single	) 🖌	00:01:00	<u> </u>	2.2	-	0 🗧	0 🗧 0	•

13.9 Then choose the "Sample Editor" on the left side.

	Subset Editor		
(	Sample Editor	)	
	Analysis		

13.10 Choose "Abs Quant".


-Step 1: Select Workflow									
Step 1. Select	WORKING W								
📀 Abs Quant	🗌 🔘 Rel Quant -	🔘 Scanning 🚽	🔿 Color Comp						
🔿 Tm	🔘 Melt Geno	🗌 🔘 Endpt Geno							
		- •							

13.11 Select the wells containing the samples, then enter the sample name and choose the sample type "Unknown". If wells contain positive control or negative control, choose the sample type "Positive Control" or "Negative Control".

- Step 2: Select Samples													
Sul	bset	Al	1 3	ampl	les			-	(D	)[8	2)(	<b>R</b> )(	R
	1	2	3	4	5	6	7		9	10	_	12	
A													
BC		<u> </u>	<u> </u>										
D													
톧													
G													
H													=
H													
_	· · · · · · · · · · · · · · · · · · ·												
Q	Quantification Sample Type												
c	Cannot show colors with multiple channels selected.												
- S1	tep 3	3: Ed	it Al	os Qu	Jant	Рго	pert	ies-					
Sa	Sample Name												
	C Negative Control     Positive Control/Calibrator     Standard Concentration     Auto Std Curve												
	ા	anda	aru	Con	cent	adl	- T		CA	atos	stu	Curv	•




13.12 Open the machine and place the samples in. Then go back to "Experiment"

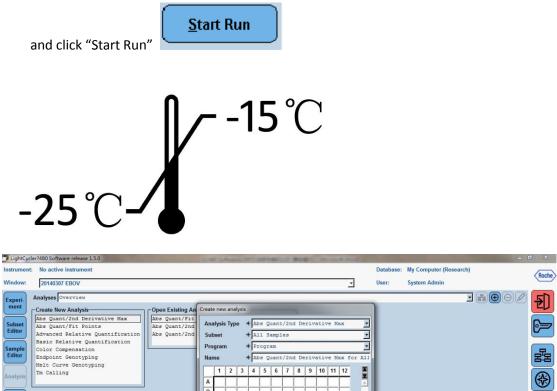

Subset	Abs Quant/2nd Derivative Max Abs Quant/Fit Points	Abs Quant/Fit Abs Quant/2nd	Ana	lysis Typ	e *	Abs	Quant	c/2nd	d De	riva	tive	e Ma	x	•		6=
Editor	Advanced Relative Quantification	Abs Quant/2nd	Sut	set	*	A11	Sampl	les			_	_		-		
	Basic Relative Quantification					_				_	-	-		-		
Sample	Color Compensation		PIO	gram		_	gram							_		
Editor	Endpoint Genotyping		Nar	ne	*	Abs	Quant	c/2nd	d De	riva	tive	e Ma	ax for	All		물물
	Melt Curve Genotyping			1 2	3	4 !	5 6	7	8	9	10	44	42	T		
Analysis	Tm Calling			1 2	3	4 .	5 0	-	0	3	10		12	X		
			A													<b>*</b>
			В													
Report			C													
$\leq$			D											-		
Sum.			E											닅		
Juin.			F		itti	-i-		Ē		i - i	-i	-i	_			$\Leftrightarrow$
			G		÷			ii	H		-h	-h	-			$\sim$
			H					-	-		-	-ł	-			
											-	_				$\otimes$
				4				<u> </u>		_	_		1			
												1	$\bigcirc$	2		
												l		$\mathbf{\circ}$		
			-	_	-	-	-	-	-	-	-	-	ОК		f	
													OR	1		13
																55
_	E					****		•								
$\square$																

Click "Calculate" button on the lower left.

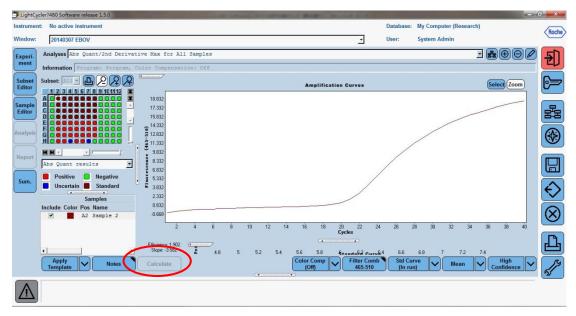

## **Interpretation of Results**

### **14 Interpretation of Controls and Clinical Specimens**

### 14.1 Controls

Test Run is Valid when all Controls Meet the Following Stated Standards.

Negative control reaction must be negative without an amplification curve at FAM channel and Ct value between 25 and 38 at VIC channel. If failed, the test run is invalid and potential sources of contamination should be identified and corrected (see Table 1).

Positive control reaction should be positive at FAM channel, Ct value must be  $\leq$ 35. If failed, obtain new reagents and re-test the run (see Table 1).

Controls	Ct value							
Controis	FAM	VIC						
Negative control	UNDET, No Ct, N/A	25~38						
Positive control	≤35							

Table 1	Inter	pretation	of	contro	١s
Table T	inter	pretation	UI.	CONTINU	13

#### 14.2 Specimen

- 14.2.1 When all controls are performed correctly, the run is valid.
- 14.2.2 A specimen is considered positive for Ebola virus RNA if the specimen has a positive amplification result at FAM channel and Ct value  $\leq$ 38 (see Table 2)<sup>2</sup>.
- 14.2.3 When Ct value is between 38 and 40 at FAM channel and Ct value is between 25 and 38 at VIC channel, the sample should be re-tested. If it is still the same result, the specimen is reported as below the detection limit or negative for Ebola virus (see Table 2).
- 14.2.4 A specimen is considered negative for Ebola virus if it does not have an amplification curve at FAM channel and Ct value between 25 and 38 at VIC channel (see Table 2).
- 14.2.5 If it has no positive amplification result at FAM and VIC channel, it is considered as no diagnosis because of some inhibition on the procedure. Repeat the whole run and re-analyze (see Table 2).

	Ct va	lue	Pocult Applycic			
	FAM	VIC	– Result Analysis			
1#	UNDET, No Ct, N/A	25~38	Below the detection limit or negative			

;
,

<sup>&</sup>lt;sup>2</sup> Assay results are for the presumptive identification of Ebola virus and intended for use as part of a multi-test algorithm to detect the presence of Ebola virus RNA. Laboratories are required to report results to the appropriate public health authorities. The definitive identification of Ebola virus requires additional testing to be performed by designated laboratories. The diagnosis of Ebola must be made based on history, signs, symptoms, exposure likelihood, and other laboratory evidence, in addition to the detection of Ebola virus RNA

2#	≤38		Positive;
3#	38~40	25~38	Re-test; if it is still 38~40, report as 1#
4#	UNDET, No Ct, N/A	UNDET, No Ct, N/A	PCR inhibition; no diagnosis can be concluded

## **15 Limitations**

- All results should be interpreted by a trained professional in conjunction with the patient's history and clinical signs and symptoms.
- Interpretation of results must account for the possibility of false-negative and false-positive results.
- Negative results do not preclude infection with Ebola virus and should not be the sole basis of a patient treatment/management decision.
- False positive results may occur from cross-contamination by target organisms, their nucleic acids or amplified product.
- Failure to follow the assay procedures may lead to false negative results.
- Improper collection, storage, or transport of specimens may lead to false negative results.
- Inhibitors present in the samples may lead to false negative results.

## **Analytical Performance Characteristics**

#### Accuracy of measurement

A series of precision studies have demonstrated high accuracy of the kit. Precision tests were performed with three lots of kits and three different concentrations of EBOV pseudovirus plasmid samples. Each test was repeated for 10 times. All CV% values are <3%. The test results are summarized in the following table:

Sample Sample concentration		CV%			CV%
Sample	(copies/ml)	Lot# 1	Lot# 2	Lot# 3	(between 3 lots)
PO	0	1.25%	1.13%	2.14%	1.55%
P1	1 x 10 <sup>4</sup>	1.54%	2.25%	2.06%	1.97%
P2	1 x 10 <sup>7</sup>	1.22%	1.57%	1.55%	1.50%

### Analytic sensitivity

Four EBOV pseudovirus plasmids were examined in the sensitivity tests. These pseudoviruses include EBOV-Z-1, EBOV-S-1, EBOV-B-1, and EBOV-C-1. The plasmid samples were diluted lineally into various concentrations. The tests were performed with three lots of kits. Each test was repeated for 3 times. Then confirming tests for the positive samples with lowest EBOV concentration were further repeated for 20 times. The results suggested that the minimum detection for all four Ebola pseudovirus is  $1 \times 10^3$  copies/ml.

## Analytic specificity

22 microorganisms and 3 human genomic DNA samples were examined with EBOV Real Time RT-PCR Kit in the specificity tests. The tests were performed with three lots of kits. Our results showed that the kit only detected Ebola virus; whereas negative EBOV detection were observed using other 22 microorganisms and human genomic DNA samples, suggesting high specificity of the kit for Ebola virus detection. The test results with 22 microorganisms and human genomic DNA samples are summarized in the following table:

	Lot # 1		Lot # 2		Lot # 3	
Sample	EBOV	Internal	EBOV	Internal	EBOV	Internal
	test	control	test	control	test	control
HCV	-	+	-	+	-	+
Influenza A virus	-	+	-	+	-	+
Influenza B virus	-	+	-	+	-	+
Enterovirus 71	-	+	-	+	-	+
Coxsackie virus A16	-	+	-	+	-	+
RS virus	-	+	_	+	-	+
Parainfluenza virus	-	+	-	+	-	+

EB virus	-	+	-	+	-	+
HBV	-	+	-	+	-	+
Human corona virus	-	+	-	+	-	+
Japanese encephalitis						
virus	-	+	-	+	-	+
Dengue virus	-	+	-	+	-	+
Streptococcus pneumonia	-	+	-	+	-	+
Human metapneumovirus	-	+	-	+	-	+
Hemophilus influnzae	-	+	-	+	-	+
Adenovirus	-	+	-	+	-	+
CMV	-	+	-	+	-	+
Epidemic parotitis virus	-	+	-	+	-	+
Staphylococcus aureus	-	+	-	+	-	+
Measles virus	-	+	-	+	-	+
Rhinovirus	-	+	-	+	-	+
Streptococcus pyogens	-	+	-	+	-	+
Human genomic DNA-1	-	+	-	+	-	+
Human genomic DNA-2	-	+	-	+	-	+
Human genomic DNA-3	-	+	-	+	-	+

### Traceability of calibrators and control material values

We have generated four EBOV pseudoviruses as positive references through vector cloning/transfection to host cells/selection of positive host cells/collection of pseudoviruses from culture media of the positive host cells. The Ct values of four EBOV pseudoviruses are summarized in the following table:

	EBOVZ	EBOVS	EBOVC	EBOVB
Ct value	16.56	16.93	17.21	16.36

We also generated five EBOV negative references from clinical samples.

Additionally, we have generated EBOV positive control, negative control, and internal control. Both positive and internal controls are pseudoviruses that were generated by the similar methods as described above. Negative control is a 0.9% saline solution that was aliquoted and stored at -20°C. Both positive and internal control were diluted before they were aliquoted and stored at -20°C. The Ct values of the positive and internal control are summarized in the following table:

	Positive control	Internal control
Ct value	16.85	16.71
Ct value after dilution	21.12	30.26