

# Medical Gas Piping Systems Safety

Intended for health workers and all personnel managing medical oxygen

Do V

Do not X

## DO REPORT ANY GAS LEAKS AS **SOON AS DETECTED**

- Report if any noise of leaking gas is detected to maintenance department for urgent action.
- Set off medical gas escape alarm.




#### DO NOT STAY NEARBY THE GAS LEAKAGES

Do not stay in the vicinity of the gas leakage because of the possibility of gas saturation in the air. asphyxiation, explosion, fire.



## DO ENHANCE VENTILATION IF A LEAK IS DETECTED

Open windows and doors to lower the gas concentration in the air.

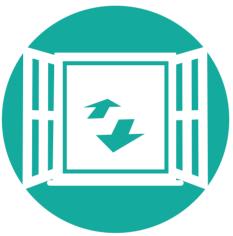


## DO NOT COVER PIPING LINEMAKERS **AND SIGNAGE**

Do not paint over piping networks to cover labeling, identification, flow direction arrows, valve box labels, etc.



#### DO REGULAR CHECKS TO THE PIPING

 Check for damage, especially serious abrasion, corrosion, leakages, flattening or mechanical damage of pipelines.

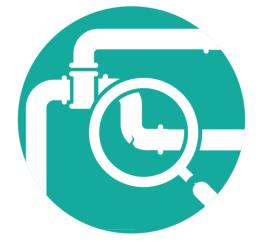

#### DO NOT USE OILY-SOAP SOLUTIONS TO CHECK FOR LEAKS

Do not use water solutions with soap that contains oil and/or animal fat. The animal fats may burn easily in an oxygen enriched atmosphere.



References:

Medical Gases, Health Technical Memorandum 02-01: Medical gas pipeline systems. Part B: Operational management. Department of Health, UK. http://www.bcga.co.uk/assets/HTM 02-01 Part B.pdf



